



PRODUCT  
BROCHURE

State-of-the-art aesthetic  
equipment backed by expertise.  
Trusted by professionals, tested  
for reliability, and designed for  
success.

LUMA  
SOLUTIONS






## About Luma Solutions

At Luma Solutions, we specialize in providing high-quality aesthetic machines designed to ensure safety and client satisfaction. With experience of over three decades in business and extensive expertise in the aesthetic industry, our team's journey began with the challenge of finding reliable equipment for our own clinics. This challenge inspired us to conduct thorough research on the best available options.

## Our Commitment to Quality

As clinic owners ourselves, we understand the importance of utilizing effective machinery. We not only design a selection of our machines in the UK, but we also use several of them in our own practices. We are committed to selling only those machines that have a high client demand for treatments, ensuring that you can maximize your profits while providing sought-after services.



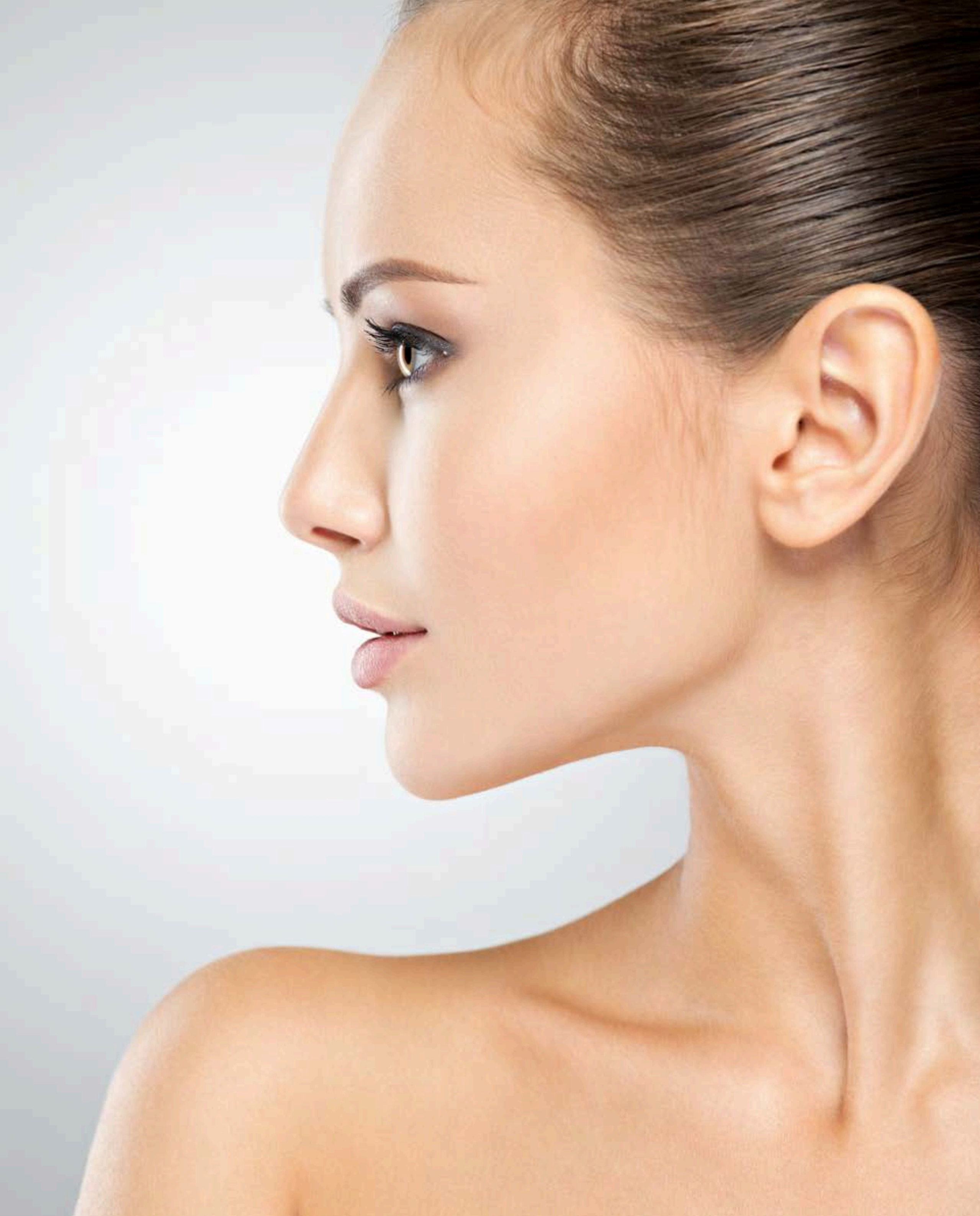

## Trusted Manufacturers & Engineering Expertise

To guarantee excellence in our products, we meticulously choose our manufacturers based on their extensive quality control measures.

Their comprehensive teams of quality check engineers and dedicated safety departments ensure that every machine meets the highest standards before it ever reaches your clinic. Additionally, our engineers conduct thorough inspections and safety checks of all machines at our UK depot to verify their performance and safety prior to dispatching them to you. This rigorous attention to detail gives us confidence that the equipment we provide is both reliable and safe.

Through extensive discussions with certification bodies in the UK and collaboration with skilled, UK-based laser engineers who have worked with leading UK laser companies, we identified crucial criteria for selecting reputable manufacturers. These criteria include valid certifications, strong reputations, and a solid safety check department.

Our engineers now partner with Luma Solutions to offer maintenance checks, servicing, and repairs, ensuring your machines operate at peak performance.



## Custom-Tailored Solutions

Our team works closely with these manufacturers, supplying detailed specifications and guidelines that allow us to tailor our machines precisely to the needs of our clients, offering unique solutions that stand out in the market

## Reliable and Affordable Machines

We are proud to present a carefully sourced selection of top quality machines that meet industry standards. Our mission is to provide you with affordable and reliable machines that enhance your practice and drive profitability.

## Comprehensive Training & Support

Additionally, Luma Solutions collaborates with experienced educators who possess a wealth of knowledge in the operation of aesthetic devices. They offer comprehensive, fully accredited device training, either at your premises or online. This ensures that you and your staff are fully equipped to utilize the technology effectively

## Customizable payment solutions

Select from Various Budget friendly options to enhance your business's revenue. At Luma Solutions we provide tailored payment plans that align with your cash flow needs Enabling you to offer exceptional skin care treatments with cutting edge beauty devices .

## Available payment options

- One -time purchase: Make a 50% deposit to secure your order then pay the remaining balance upon delivery.
- Simple rental: easily rent a premium device and connect with us at the end of your rental for the next steps .
- Flexible financing: explore various payment plans with our trusted finance partners.



# LUMA SCULPT



**5000W STRONG POWER 4 HANDLES  
WORK SIMULTANEOUSLY**

**+25% Muscle MORE**

**-30% Fat LESS**

**COMBINED 30-MIN.TREATMENT**






**RF**

**High Intensity Focused Electro-Magnetic**

**-30% Fat**

**LESS**

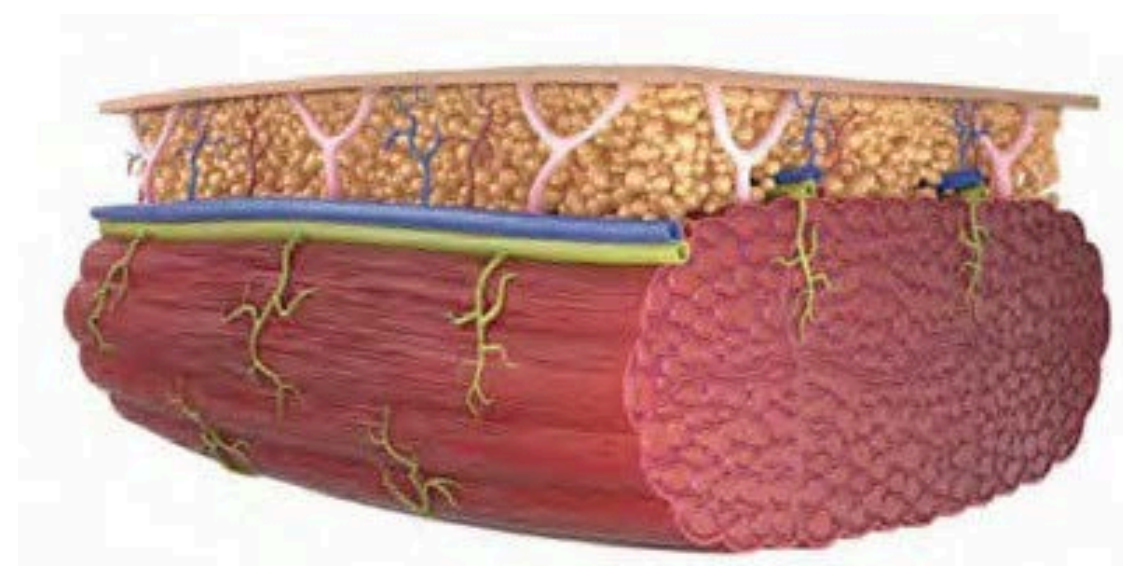

**+25% Muscle**

**MORE**





EMS machine is based on an applicator simultaneously emitting synchronized RF and High Intensity Focused Electro-Magnetic energies.

Due to the radiofrequency heating, the muscle temperature quickly raises by several degrees. This prepares muscles for exposure to stress, similar to what a warm up activity does before any workout. In less than 4 minutes, the temperature in subcutaneous fat reaches levels that cause apoptosis, i.e. fat cells are permanently damaged and slowly removed from the body. Clinical studies showed on average a 30% reduction in subcutaneous fat.\*


Bypassing the brain limitations, High Intensity Focused Electro-Magnetic energy contracts the muscle fibers in the area at intensities that are not achievable during voluntary workout. Extreme stress forces the muscle to adapt, resulting in an increase in the number and growth of muscle fibers and cells. Clinical studies showed on average a 25% muscle volume increase.\*



# Mechanism of muscle contraction

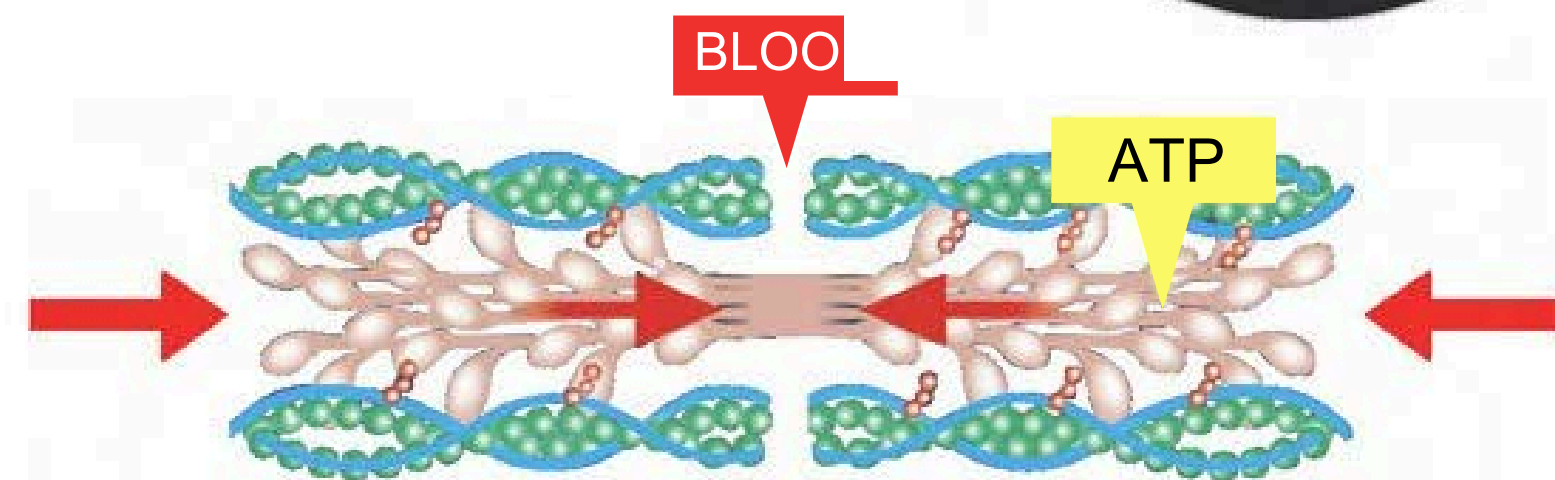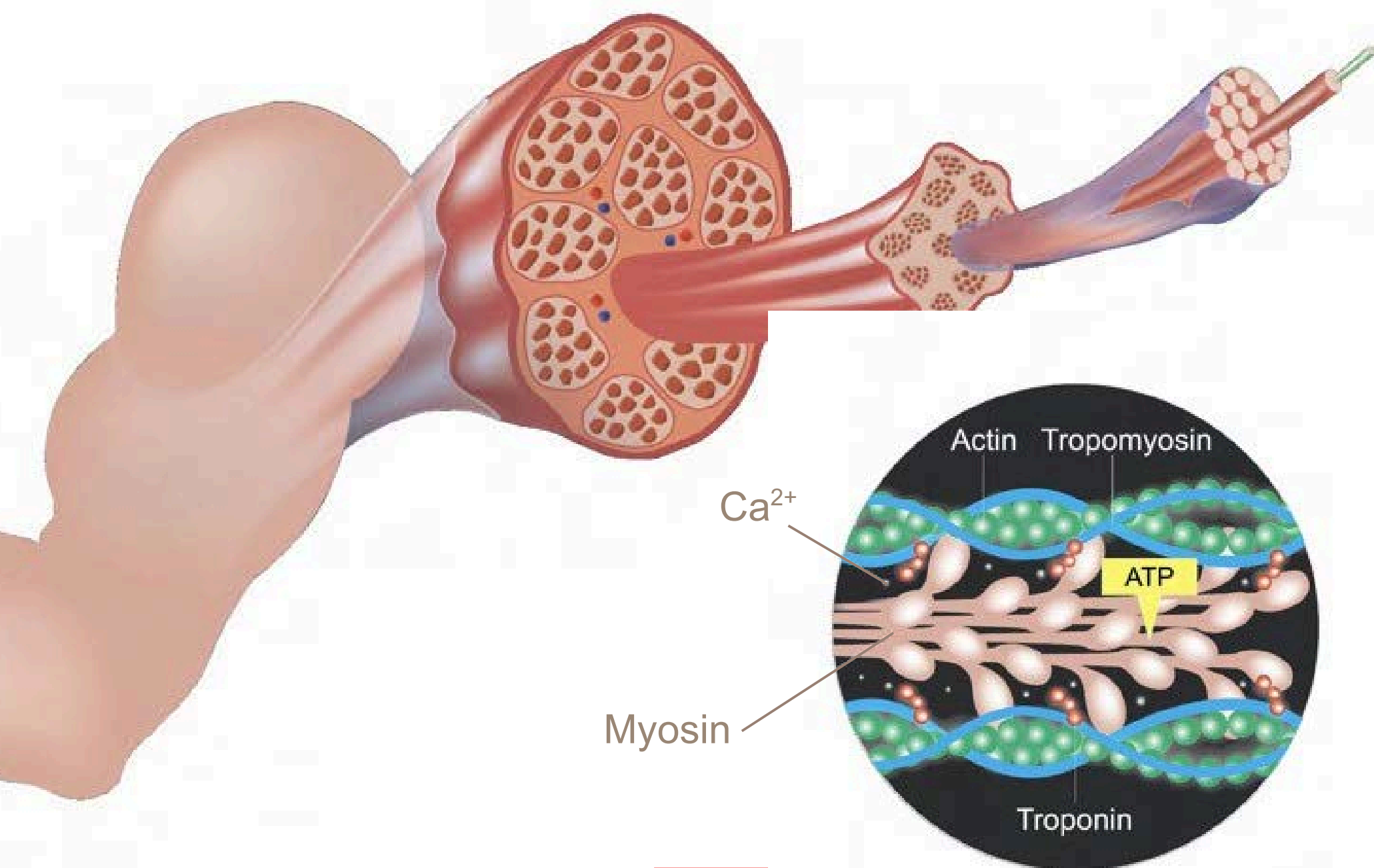


Contraction period

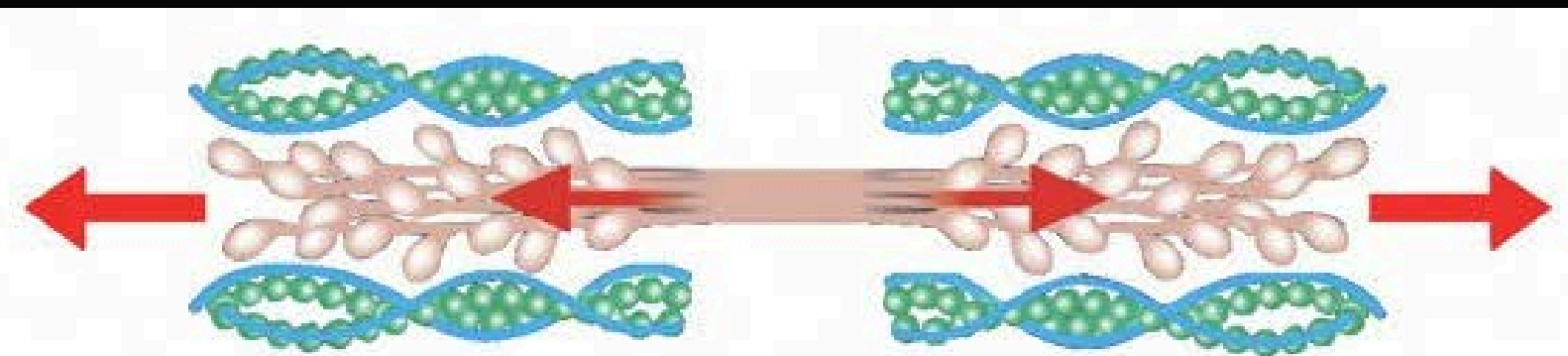

Relaxation period

Nerve impulses set off a biochemical reaction that causes myosin to stick to actin

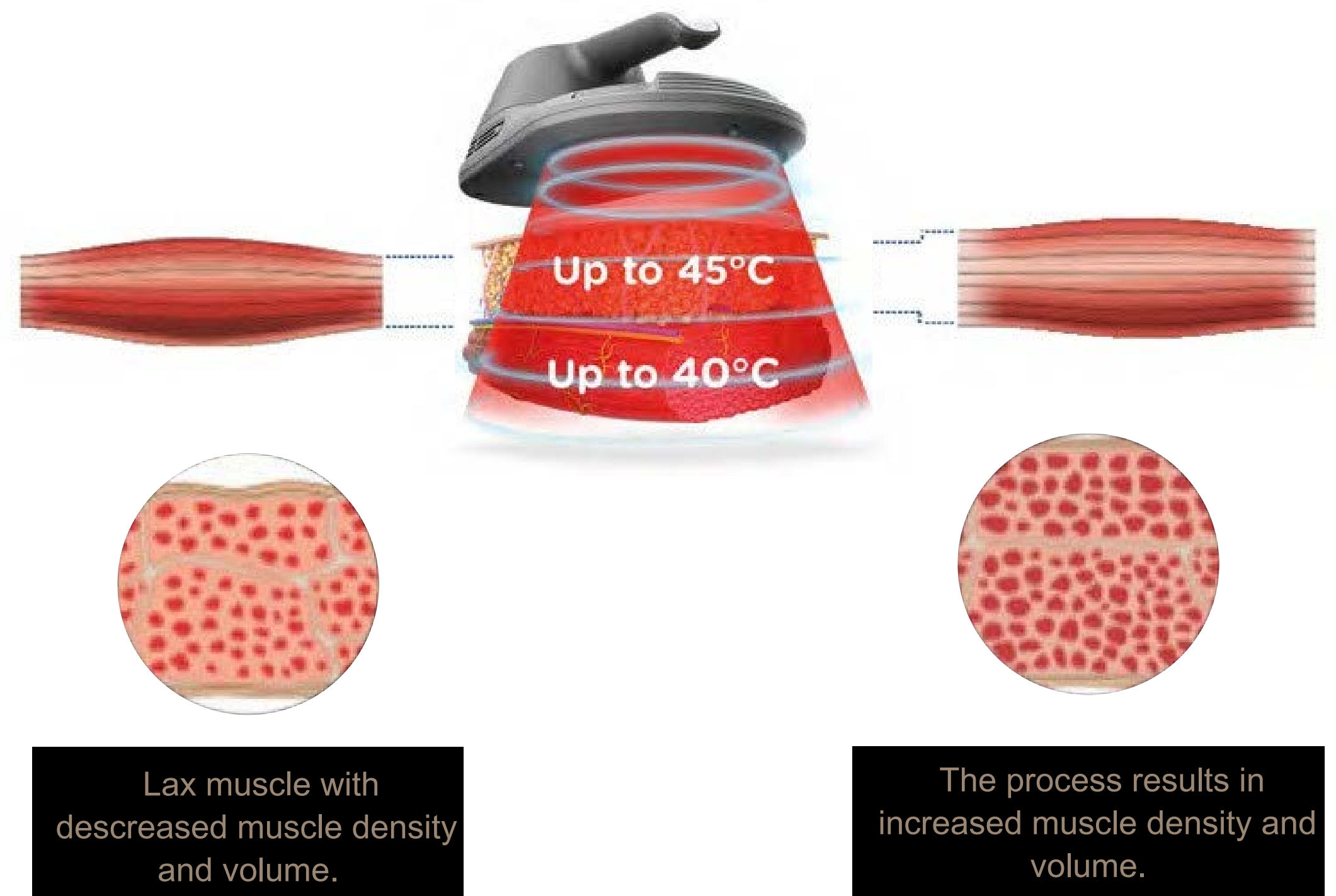

EMS machine induces very strong (supramaximal) muscle contractions, not achievable through workout. When exposed to these strong contractions, the muscle tissue is forced to adapt to such extreme condition. The muscle starts remodelling its inner structure by multiplication of fibers and their growth. The High Intensity Focused Electro-Magnetic uses a specific range of frequencies that does not allow muscle relaxation between two consecutive stimulations. The muscle is forced to remain in contracted state for multiple seconds. When repeatedly exposed to these high load conditions the muscle tissue is stressed and is forced to adapt. Recent studies reported that on average 15%- 16% increase in abdominal muscle thickness was observed in treated patients one to two months after High Intensity Focused Electro-Magnetic treatments.

## FLAT HANDLE



ABDOMEN

BUTTOCKS

THIGHS


BACK

## CURVED HANDLE



ARMS

CALVES

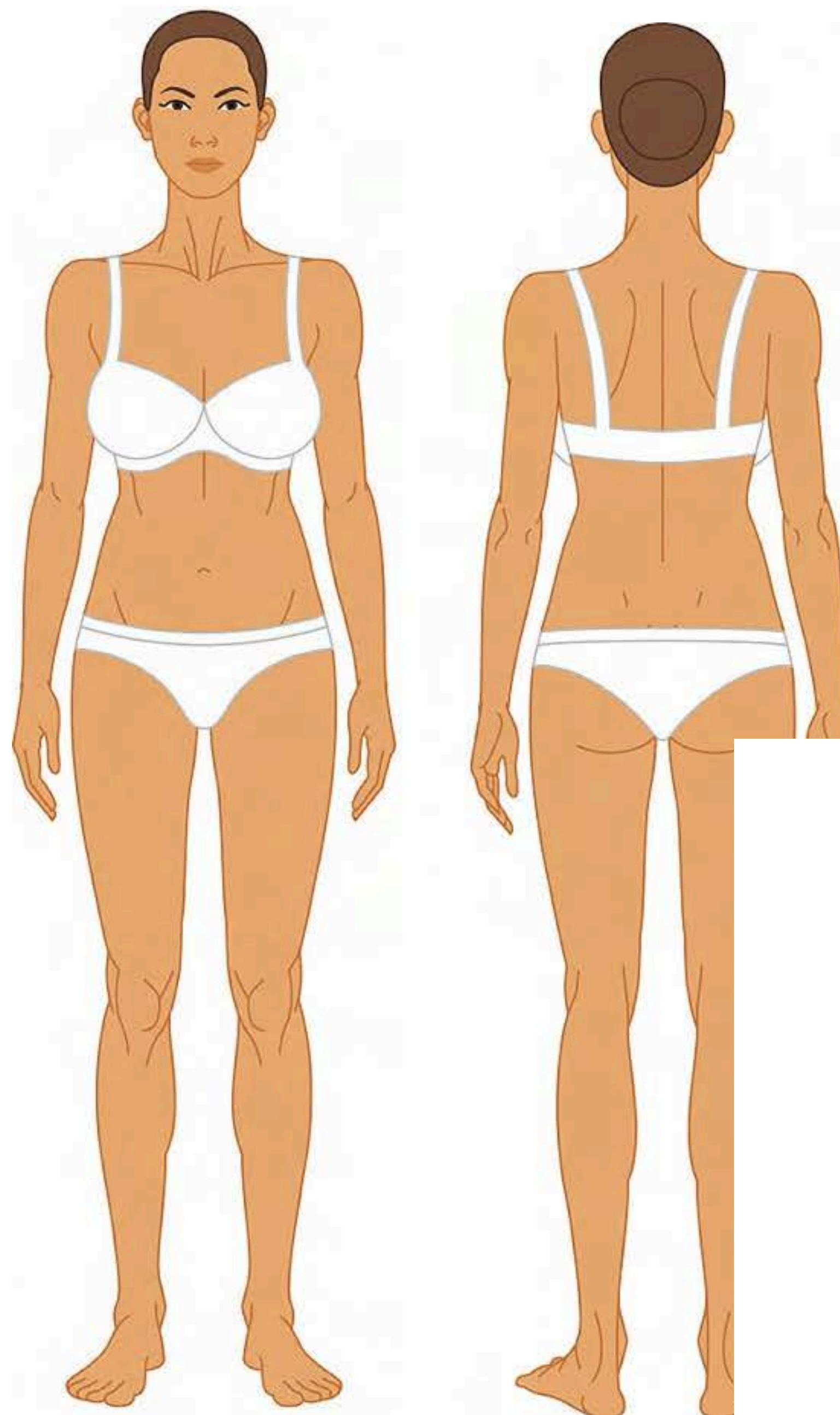


## PELVIC FLOOR HANDLE

Pelvic floor handle works to relieve incontinence by stimulating and strengthening the underlying muscles of the pelvic floor. During the treatment, you sit on a specially designed handle which delivers electromagnetic energy to the pelvic floor muscles, forcing them to contract. Over the course of your treatment, these contractions strengthen the weakened muscles and restore neuromuscular control of the bladder.





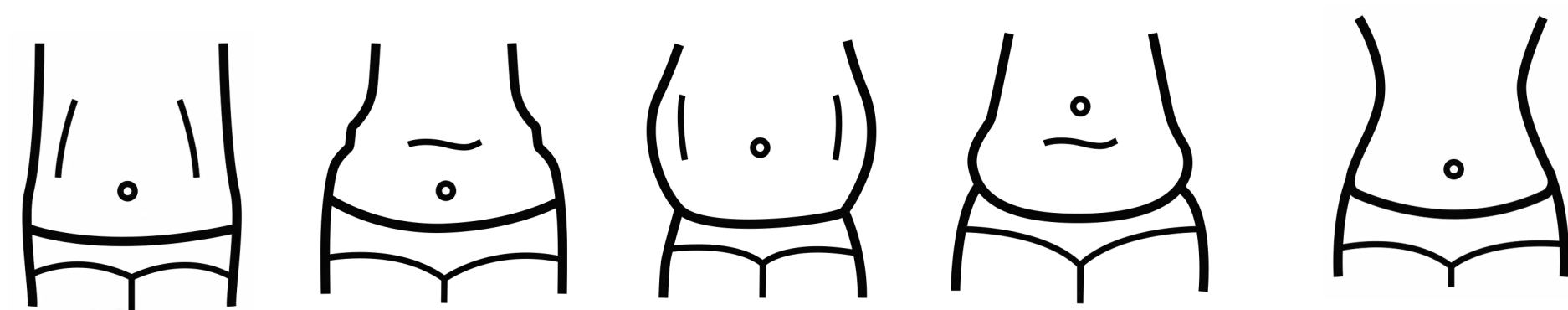


H Type

Folding Type


Fat Type

After Childbirth

Build abdominal muscles  
Perfect Firm abs  
and V line abs

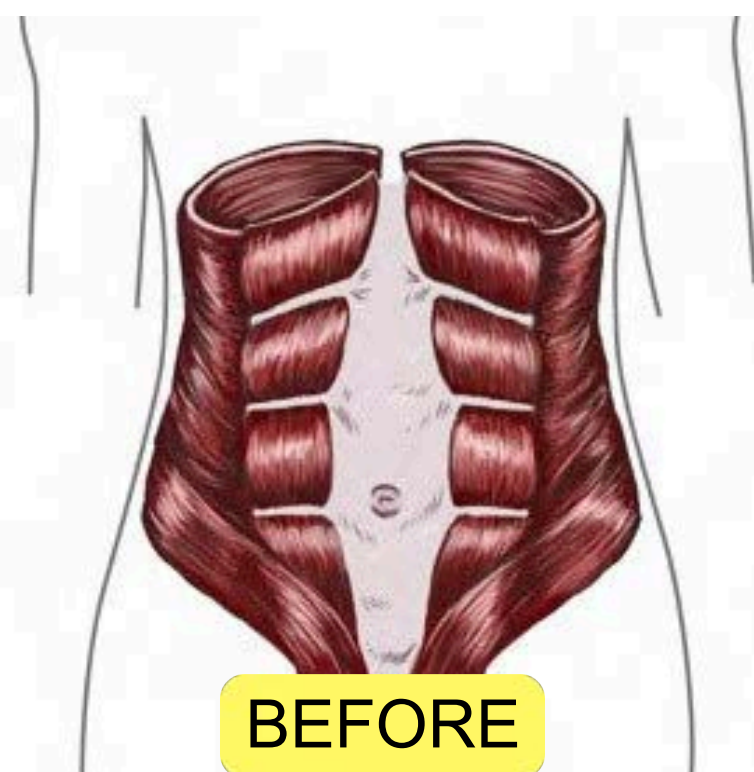

BEFORE

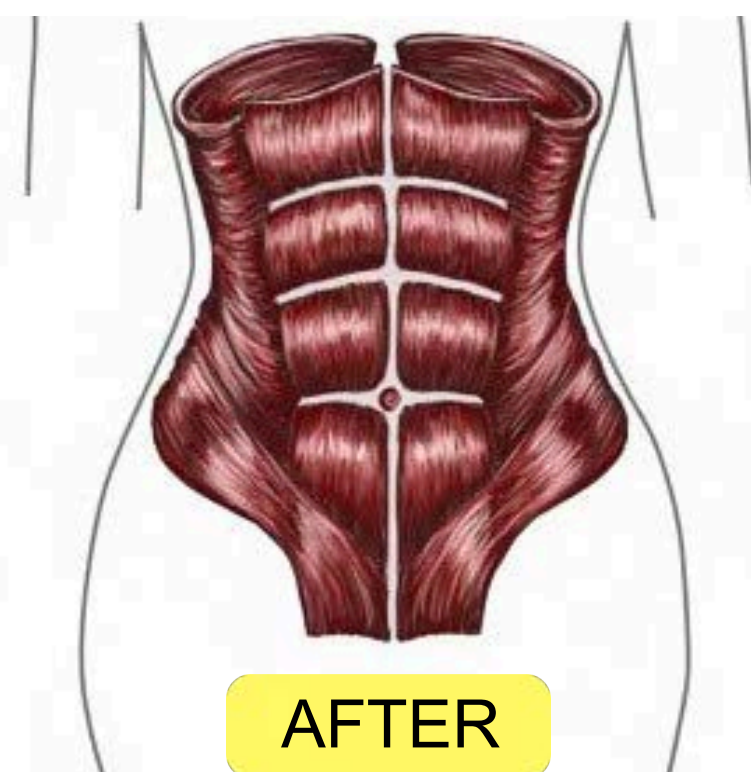


AFTER


The EMS machine works on the abdomen area contracting and tightening the muscles to both burn fat and sculpt the area. This treatment can help you improve the muscle definition that you are already seeing from your fitness routine. It is also ideal for addressing that stubborn inch of fat on your stomach that just won't go away.



# FOR THE ABDOMEN

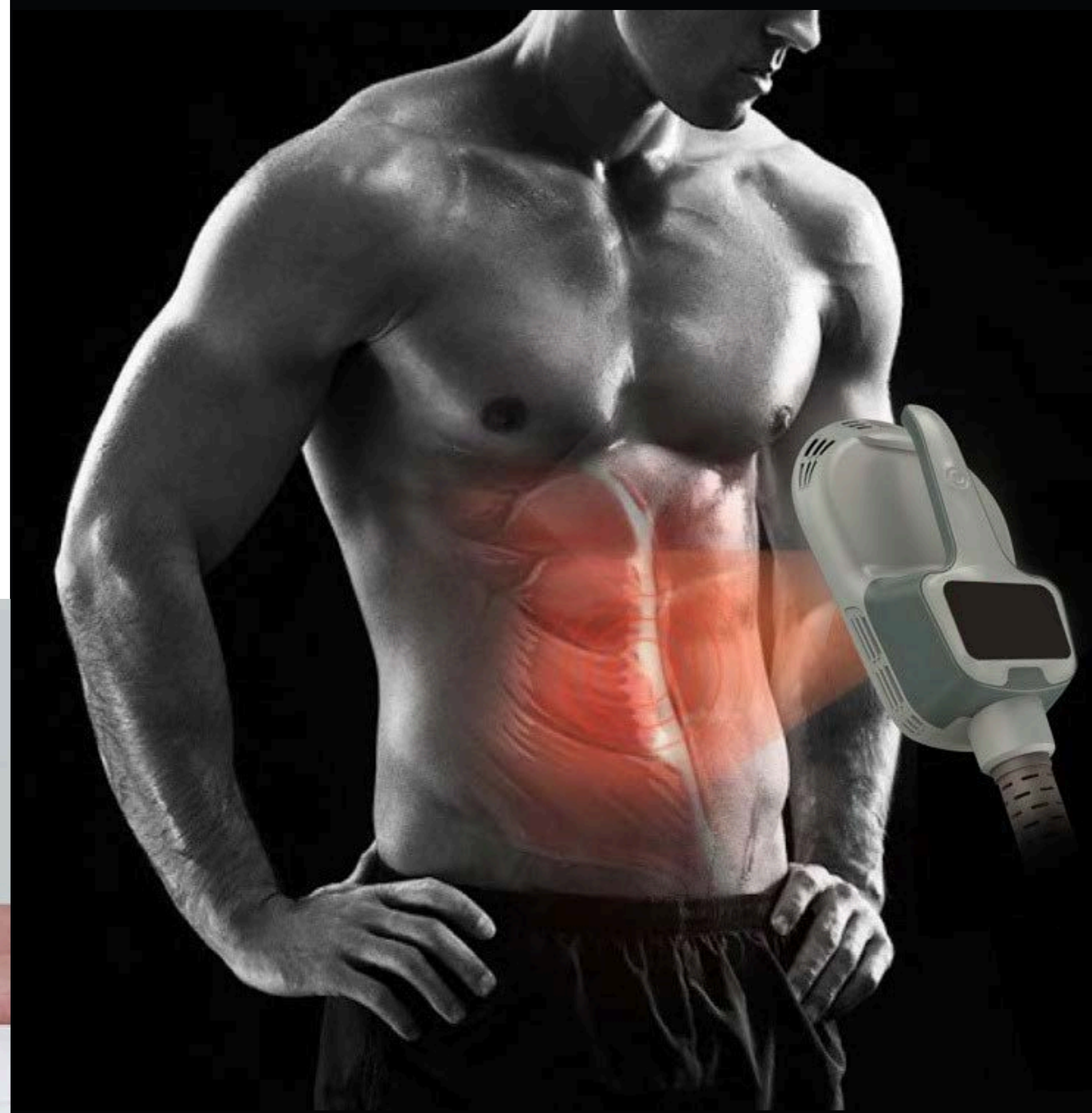



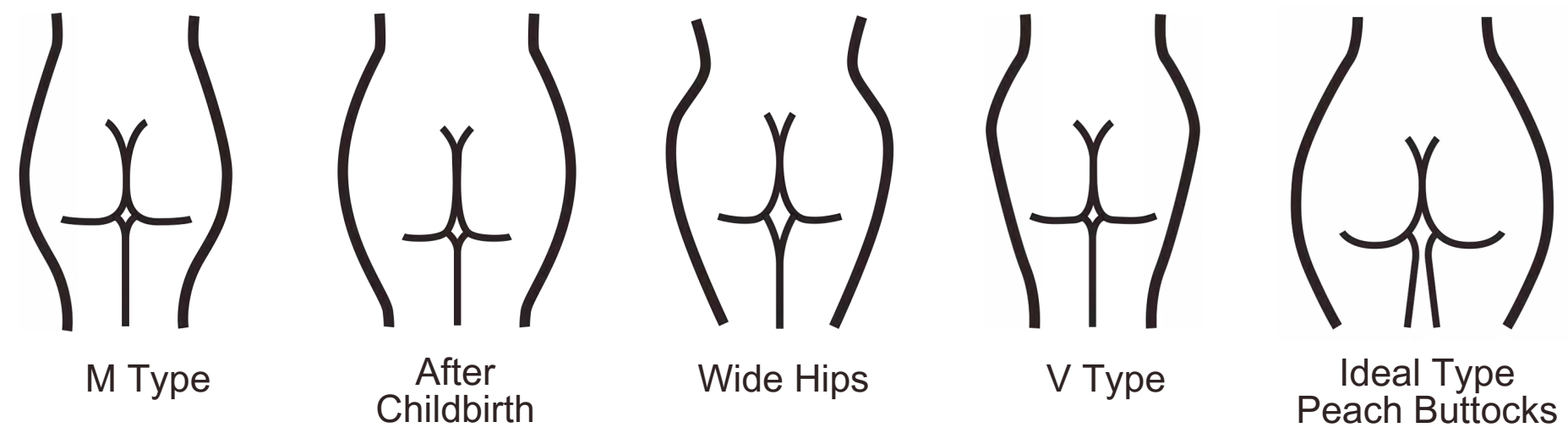


# FOR THE BUTTOCK



EMS machine works on the buttocks as well, using the same supramaximal muscle contractions as with the abdomen area. Many patients opt to try this treatment in place of an invasive Brazilian butt lift. High Intensity Focused Electro-Magnetic technology delivers the world's first noninvasive butt lift procedure. This treatment can give you the curvature, tone, and lift that you desire. It is an alternative to the Brazilian Booty Lift procedure and provides a defined shape instead of adding excess fat to the area.





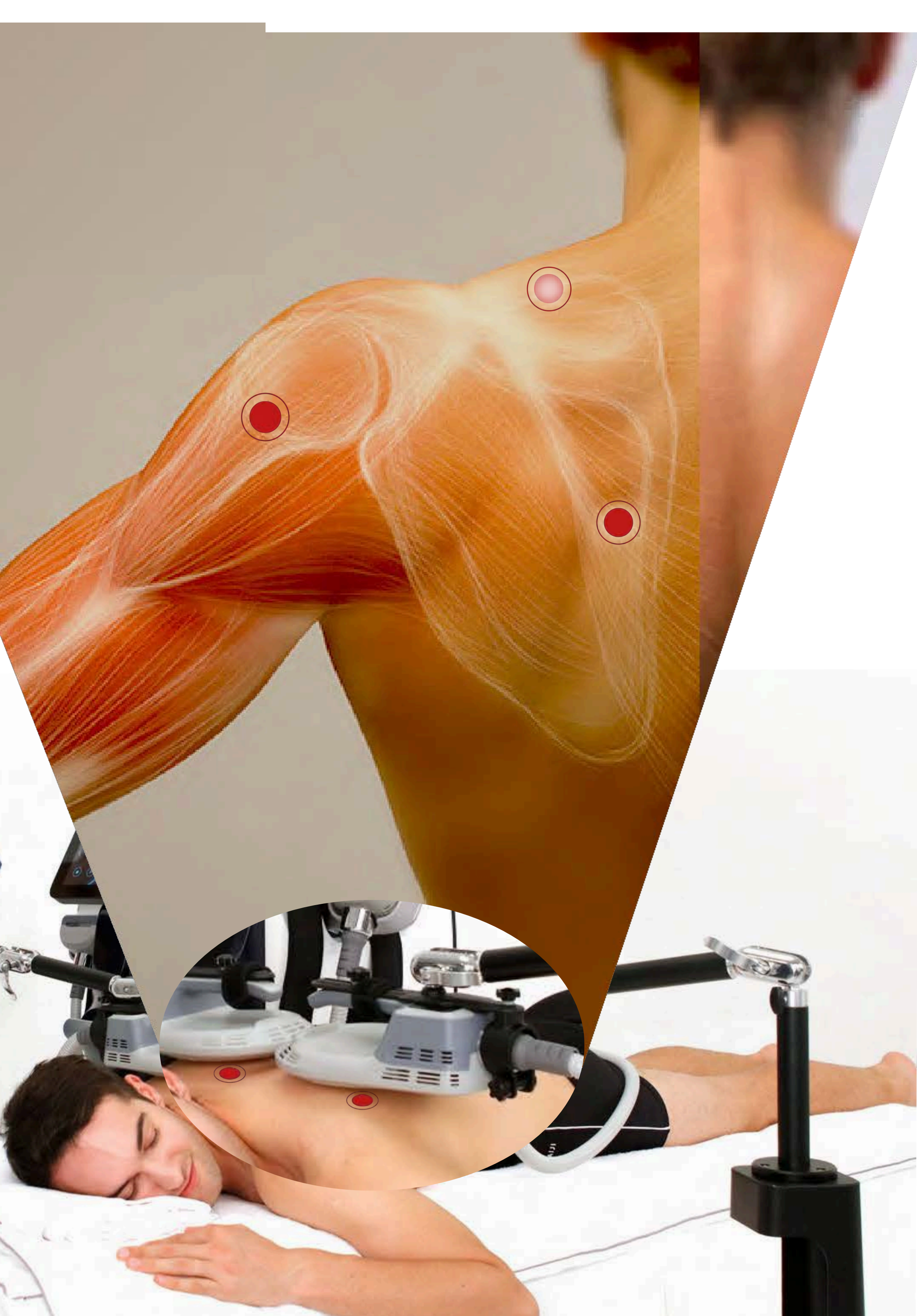

## Dedicated multi-directional handle fixed arm

Multi-directional fixed support arm can be adjusted freely at any angle of 360°

The multi-directional fixed arm greatly facilitates the treatment of various parts of the body, and perfectly solves the problem that the handles of the shoulders, handles, shoulder blades and other parts are not easy to fix. Users can easily enjoy the treatment of muscle gain and fat reduction in various parts of the body.





# Intelligent Operating System

## GENDER SELECTION


Male | Female

## TREATMENT AREA SELECTION

Buttock | Abdomen | Thigh | Arm | Calf

## WORKING MODE SELECTION

Hiit | Strength | Muscle Building | Shape Up | Fat Burning

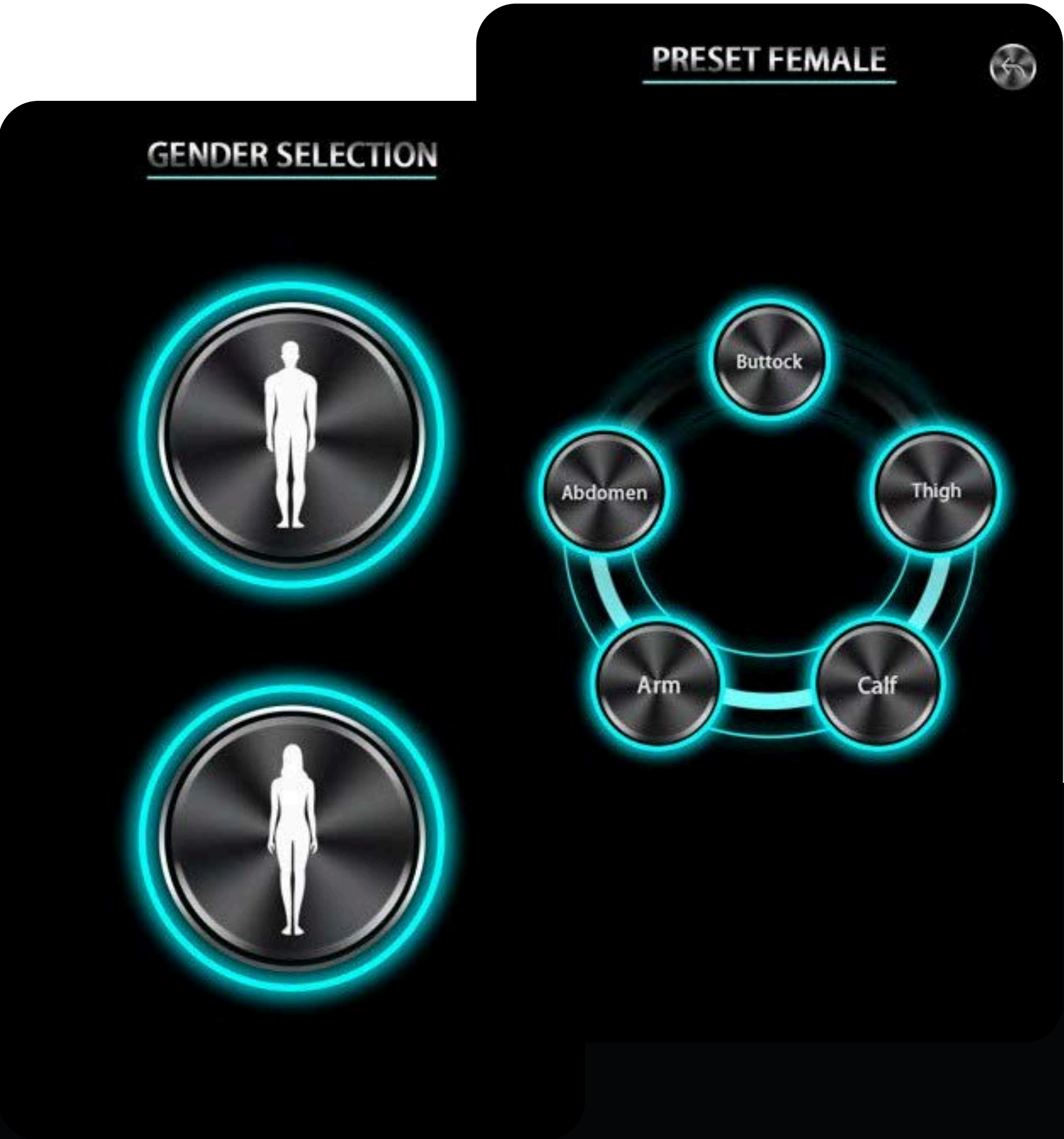

The new intelligent operating system can automatically match different treatment parameters according to different treatment parts; at the same time, it is equipped with five different working modes to directly match the corresponding treatment parameters according to the treatment needs of customers. Extremely easy to operate



SPECIFICATIONS

Frequency:	1-150hz
Power:	5000w
Intensity:	7Tesla
Screen:	15inch tough screen
Pulse width:	310μs±10%
Cooling:	Air Cooling
Voltage:	AC220V±10% 10A 50HZ , 110v±10% 10A 60HZ