

**F R E E D O M<sup>®</sup>**



**K N E E**



## Instrumentation Guide



# INTUITIVE DESIGN

Developed to naturally follow and logically support surgical flow.



**Femoral A/P sizing guide** allows for rotational pin placement referencing the **5-in-1 cutting block** without repositioning.

Rapid-connect **pinning system** features pinslots specially designed to increase precision and decrease jig pullout.



**5-in-1 cutting block** allows for medial/lateral placement and anterior/posterior adjustability to accurately assess femoral component fit.

Both standard and optional cuts are available on the **distal femoral cutting guide** without repositioning, with additional +/- 2mm and 1mm slot options for more precision.



Uncomplicated **tibial cutting guide** enables quick gross adjustment and fine-tuning of the proximal face cut in millimeter increments.



Clear instrument labeling for intraoperative alignment and precision.




# EFFICIENT APPROACH

Strategically designed to minimize steps, while streamlining and supporting the entire surgical process.

One-piece **femoral A/P sizing guide** (no operating room assembly required). Reversible to accommodate both right and left orientation without the need for multiple instruments.



Reversible **distal femoral cutting guide** for both right and left knee.

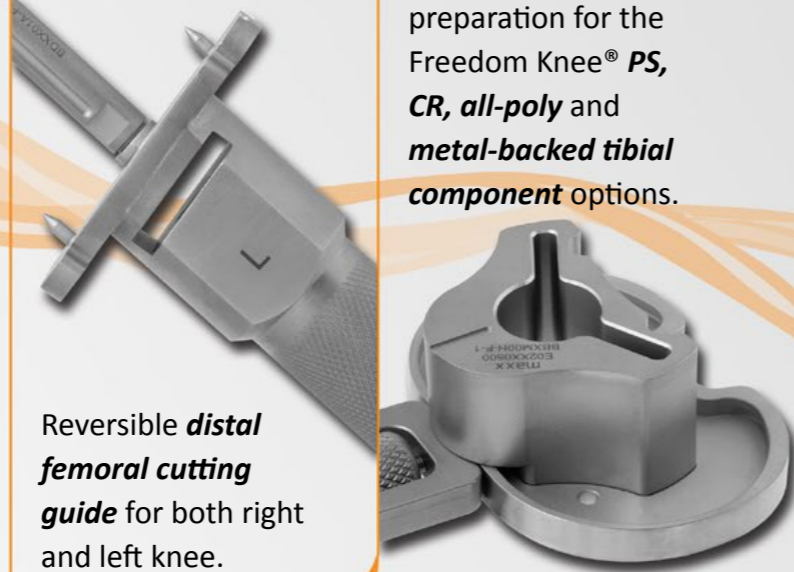

The same instruments accommodate tibial preparation for the Freedom Knee® **PS, CR, all-poly** and **metal-backed tibial component** options.

**Distal femoral cutting guide** is fixed at 6° of valgus to allow for a reproducible distal femoral face cut with lower error rates, with 4° and 8° options for more complicated anatomy.



# CONSISTENT RESULTS

High performance instruments designed to inspire confidence and deliver reproducible results.



Anterior referencing **femoral A/P sizing guide** designed for 3° and 6° of external rotation of the femoral component allowing for one jig adaptation for varus and valgus knees.




**Tibial broach** is non-serrated to preserve bone.


**Tibial cutting guide** is fixed at 3° posterior slope to the mechanical axis of the tibia in the sagittal plane, with 0° and 6° options for actively managing flexion.



# INSTRUMENT TRAY SYSTEM

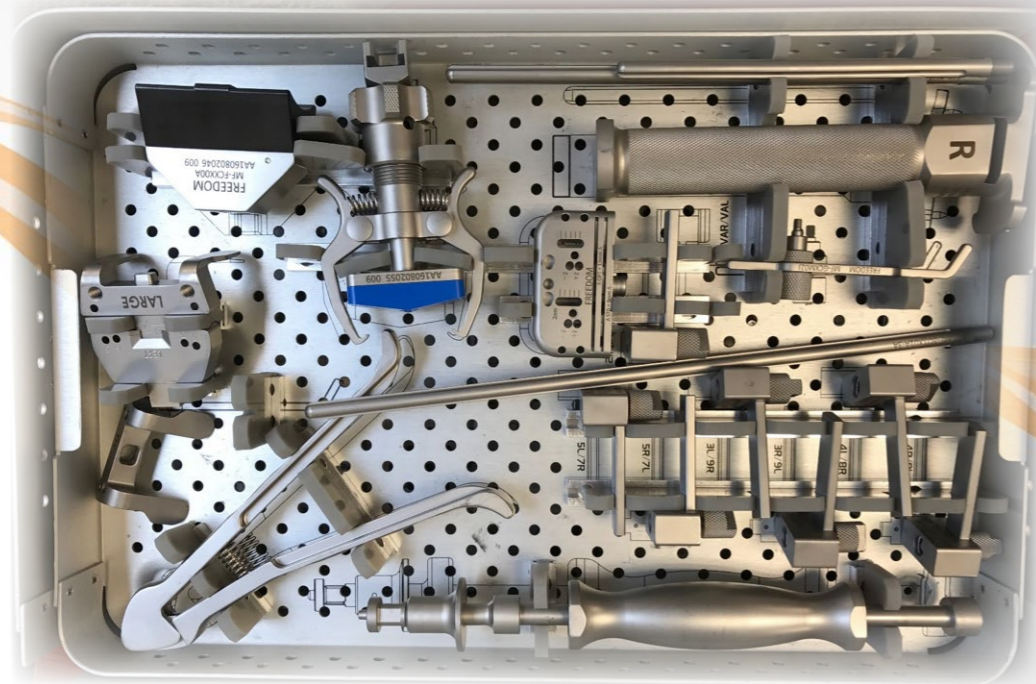

***Femoral top tray***

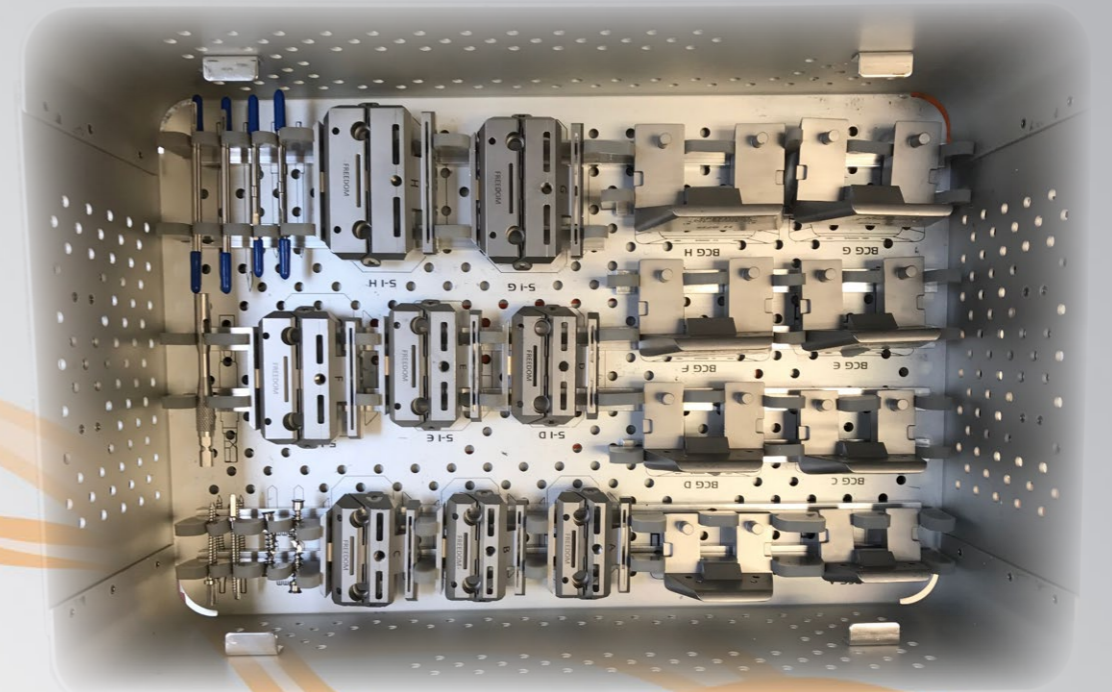


***Femoral bottom tray***

# INSTRUMENT TRAY SYSTEM




***Trial top tray***

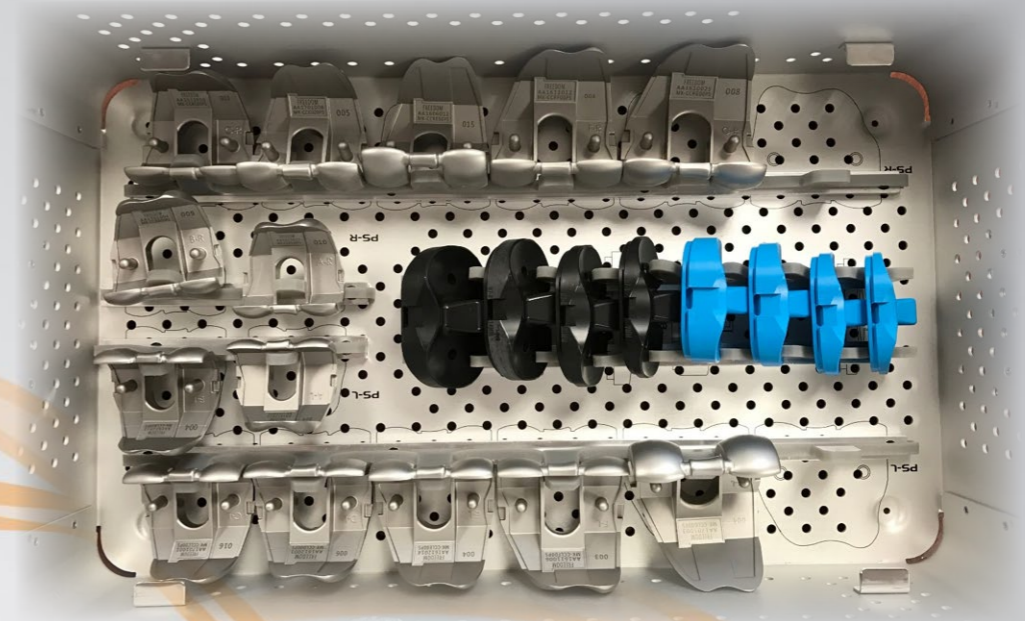


***Trial bottom tray***

# INSTRUMENT TRAY SYSTEM

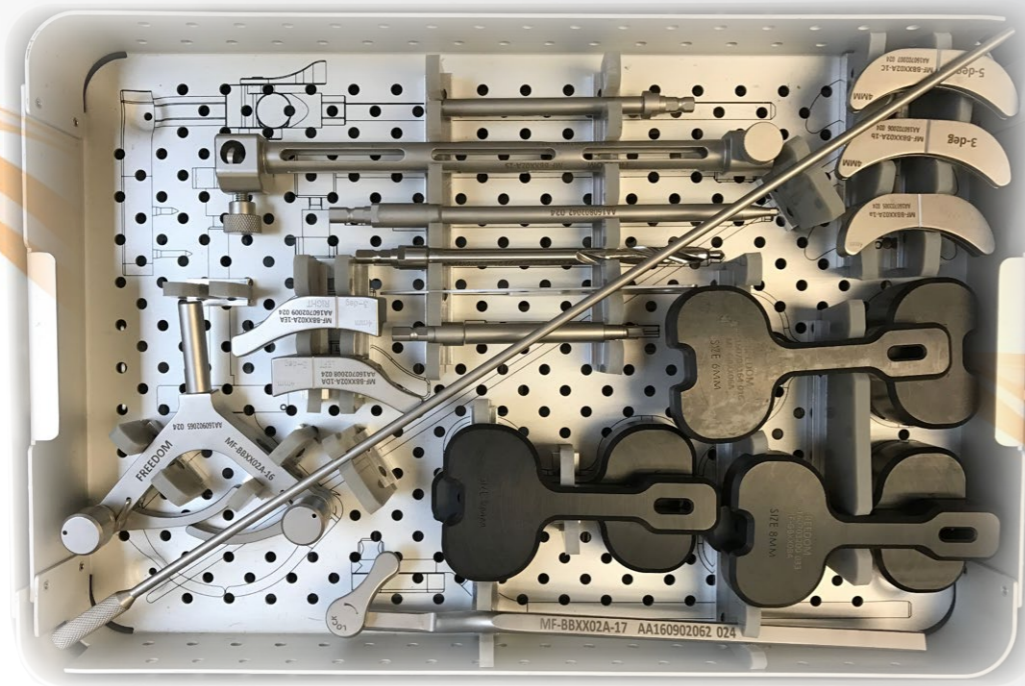

***Tibial top tray***

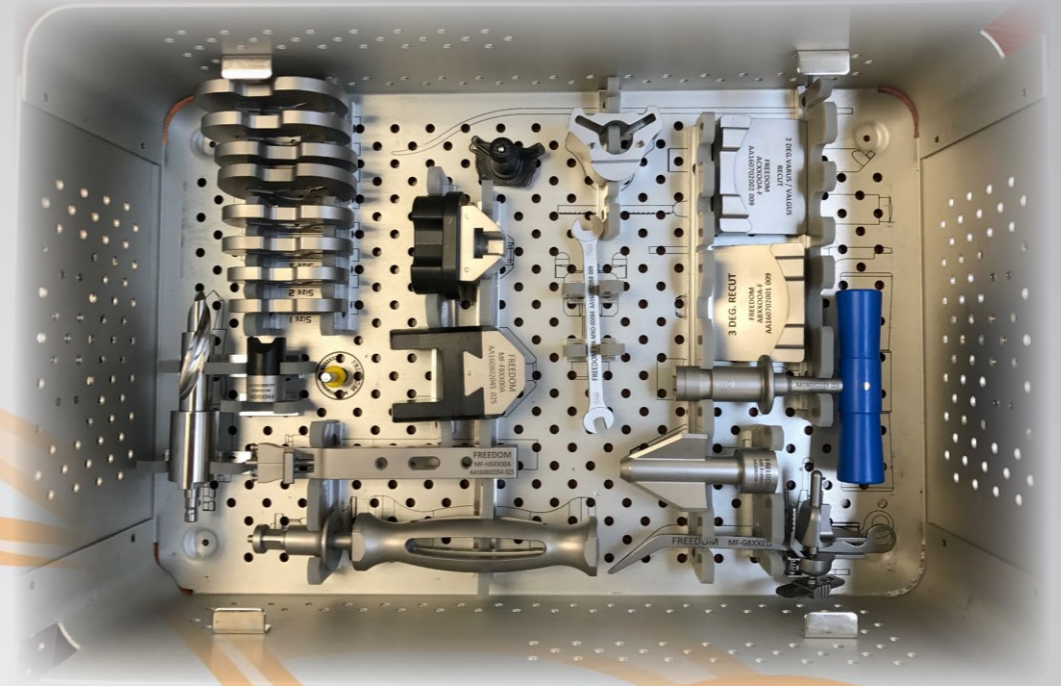

***Tibial bottom tray***

# INSTRUMENT TRAY SYSTEM



*Patella tray*

## STREAMLINED

significant reduction in the number of trays and quantity of instruments needed.

## INTUITIVE

instruments that are easy to master and use.

## LOGICAL

instrument tray layout representative of surgical steps.

## PRECISE

highly reproducible results while accommodating intraoperative variability.

## ACCOMMODATING

cross-platform flexibility across The Freedom Knee CR, PS, all-poly and metal-backed options.

**The Freedom Total Knee® System maximizes component interchangeability to enable simpler implantation, reduce procedure time, facilitate minimally invasive surgical approaches and decrease the time required to expertly manipulate the system components.**

**BETTER PERFORMANCE,  
IMPROVED PATIENT OUTCOMES.**



# Pursue Life™

For more information about Freedom Knee®  
please contact your local representative.

## Freedom Total Knee® System (CR and PS)

Rx only



Maxx Orthopedics, Inc.  
531 Plymouth Road, Ste. 526  
Plymouth Meeting, PA 19462 USA



RMS UK, Ltd. 28 Trinity Rd  
Nailsea, Somerset BS 484NU UK

CE 0050



Carefully read all instructions and be familiar with the surgical techniques prior to use.

Please see the package insert for complete device description, product selection information, indications, contraindications, precautions, adverse effects, warnings, materials, sterilization and patient guidance associated with the Freedom Total Knee® System.

**CAUTION:** THIS DEVICE IS RESTRICTED TO SALE BY OR ON THE ORDER OF A LICENSED PHYSICIAN

**WARNING:** THIS DEVICE IS INTENDED FOR CEMENTED USE ONLY

Freedom Knee® is manufactured by Maxx Orthopedics, Inc. FREEDOM and FREEDOM KNEE are Registered Trademarks of Maxx Orthopedics, Inc.

© 2012 Maxx Orthopedics. All rights reserved. Updated January 2012.

[maxx](http://www.maxxmed.com) www.maxxmed.com